Spectris Solaris® EP

MEDRAD MR

MR Injection System



"Easy to operate.
Shortens time
and increases
productivity"



"The people and products you rely on. The name you trust."

Performance. For life. Since 1964, MEDRAD has been developing products that clinicians worldwide rely on to provide outstanding performance every time they're used. Performance that helps physicians diagnose patients more accurately and, ultimately, deliver quality patient care.

You will find that same level of performance – along with the high level of customer support you've come to expect from MEDRAD - in the new Spectris Solaris EP Magnetic Resonance Injection System.







MRI Solutions You Trust.

MEDRAD offers a comprehensive array of MR compatible products, including injection systems, infusion systems, monitors and coils. MEDRAD's products known for their safety, performance and reliability are backed by our world class customer support.



The Leader Advances

MEDRAD takes the performance of the world's leading MR power injection system to new heights with Enhanced Performance capabilities, 3T compatibility and uncompromised ease of use with even more flexibility than before.

Ease of Use

- Optimized color touch-screen with few keystrokes.
- Six user-programmable phases for added flexibility.
- Independent Keep Vein Open (KVO) allows more time to focus on the patient.
- Larger 115 ml saline syringe allow for longer KVO & multiple flushes.

Reliability

- Longer battery life with increased capacity.
- Confidently backed by MEDRAD, with trusted reputation for support and unmatched reliability.


Safety

- Fiber Optic technology enables direct, reliable communication.
- Designed for use in all MR Scanner field strengths up to and including 3T.



"I wish everything worked like my MEDRAD."

"Very reliable and user-friendly"

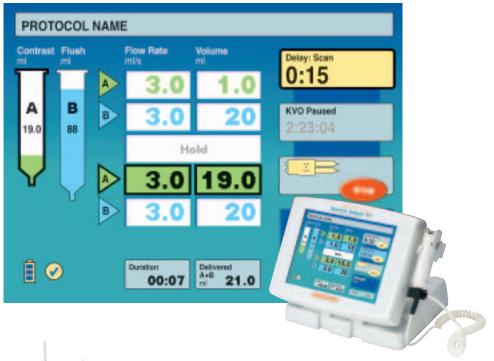



Contrast-efficient disposable system, maximizing contrast delivery

- 65 ml contrast and 115 ml saline syringes are designed as an integral part of the total fluid delivery system.
- Design of low pressure tubing eliminates dead space in the "T" connection that can waste contrast.
- The clear barrel design with molded FluidDots[®] help detect the presence of air in a syringe.



"The display is easy to read and there are less pages to go through for programming."

Flexibility

The Integrated Continuous Battery Charger (iCBC) Option

Add the iCBC Option to your MEDRAD Spectris Solaris EP and gain the ultimate in power flexibility.

- Convenient design maximizes operator efficiency by not having to change the battery.
- Easily switch between battery power and iCBC in seconds for added power flexibility.

 Flexible installation options of either in-room or out-of-room for added convenience.

Backed by MEDRAD reliability, support and service

- · Legendary product reliability
- Unmatched after-sales support
- Knowledgeable and reliable technical service
- Ask about MEDRAD's Preventive Maintenance™ Programs and Extended Warranties









MEDRAD reserves the right to modify the specifications and features described herein, or discontinue manufacture of the product described at any time without prior notice or obligation. Please contact your authorized MEDRAD representative for the most current information.

MEDRAD, Inc. is ISO 9001/ISO 13485 Certified.

MEDRAD, Spectris Solaris and FluiDots are federally registered trademarks and Performance. For life. and Preventive Maintenance are trademarks of MEDRAD,



© 2004 MEDRAD, Inc. All Rights Reserved.

SSEP NA BA 203166 Rev A

System Specifications

Components **Dimensions**

Control Room Unit 11.995" (30.46 cm)

W (screen up)

11.80" (29.97 cm) W Base Unit

(screen up)

10.922" (27.91 cm) H (screen & base)

9.25" (23.5 cm) H (screen up)

10.5" (26.67 cm) D

2.1" (5.3 cm) W **Battery Pack**

> 4" (10.2 cm) W 5" (12.7 cm) H 8.2" (20.8 cm) H 11.5" (29.21 cm) D

Battery Charger with Bracket

1.59" (4.03 cm) H 3.05" (7.74 cm) W

5.08" (12.90 cm) D

Scan Room Unit 52.25" (132.71 cm) H 19.25" (48.89 cm) W

21.5" (54.61 cm) D

Electrical

Electrical 100-240 VAC

50-60 Hz Requirements

250 VA

Unit < 100 microamperes Electrical Leakage

Patient < 10 microamperes

Disposable Syringe Kit for Spectris Solaris EP

SSQK 65/115vs Catalog Number

Contents 65 ml Qwik-Fit Syringe®

Disposable for Contrast

Media

115 ml Qwik-Fit Syringe® disposable for Saline

One Large Spike for Saline

One Small Spike for Contrast

Media

96" Low Pressure Connector Tube with

T-connector and Check Valve

System Capabilities

Volume:

0.5 ml to maximum syringe Syringe A

volume in:

0.1 ml increments between

0.5 and 31 ml

1 ml increments for 31 ml and above

Syringe B 1 ml to max. syringe

volume in:

1 ml increments

Flow Rate (Programmable)

0.01 to 10 ml/s in:

0.01 ml/s increments between .01 and 3.1 ml/s

0.1 ml/s increments between 3.1 and 10 ml/s

KVO (Programmable)

15 seconds

20 seconds

0.25 ml pulsed every 30 seconds (default) 45 seconds

60 seconds 75 seconds

Pressure Safety Limit

Factory set not to exceed

325 psi (2240 kPa)

1 to 300 seconds in Delay

1 second increments

Pause Phase 1 to 900 seconds in

1 second increments

Injection Capabilities 6 phases per protocol Storage Capacity

32 protocols of up to

6 phases each

Protocol and user configuration memory is maintained when system

power is turned off



The Malcolm Baldrige National Quality Award is our nation's premier award for performance excellence and quality achieve-ment. Established in 1987 and named after the former Secretary of Commerce, the

Baldrige Award recognizes exemplary achievements in seven areas: leadership, strategic planning, customer and market focus, measurement, analysis and knowledge management, human resource focus process management and business results. The U.S. Commerce Department's National Institute of Standards and Technology (NIST) manages the Baldrige National Quality Program in close coopera-tion with the private sector. The award is traditionally presented by the President of the United States in a special Washington, D.C. ceremony.

MEDRAD, Inc. One Medrad Drive Indianola, PA 15051-0780 USA

412-767-2400

www.medrad.com/spectrissolaris

For more information: info@medrad.com

Customer Service/Orders 1-800-MEDRAD-1 (1-800-633-7231)

Customer Service FAX (412) 767-4120

International FAX (412) 767-4128







